

How to test for Fentanyl

Powders & Pills guide

Step 1: Prepare Sample

Crush Your Pill

- Make sure your pill or powder is crushed finely into a uniform consistency




Step 2: Choose a Testing Method

Test Everything

- Pour powder into a cup
- Use 1/2 cup or 100ml of water per gram of material
- Mix until fully dissolved

Option 1 (Most Accurate)

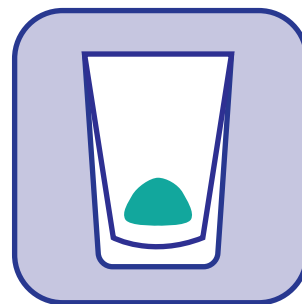

Test Bag Residue

- Make sure the contents inside the bag are a uniform powder
- Dump the contents into another container
- Fill the empty bag with water and mix if necessary


Option 2



Test a Scoop

- Put one micro scoop of substance into a small cup
- Mix 1 tsp or 5ml of water for each scoop of substance

Option 3

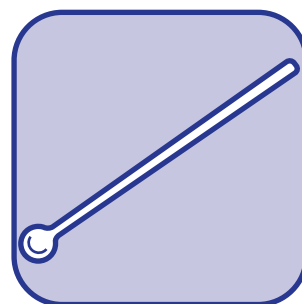

Step 3: Testing

Use the Test Strip



- Place the test strip wavy side down in the water for 15s. Make sure the water is up to the line
- Remove strip from water and place it on a flat surface for 1-2 minutes



Step 4: Results

Read the Results

- A test **POSITIVE** for fentanyl is indicated by **ONE RED LINE**
- A test **NEGATIVE** for fentanyl is indicated by **TWO RED LINES**
- Remember, no test is 100% accurate, & a negative test does not necessarily mean you are safe

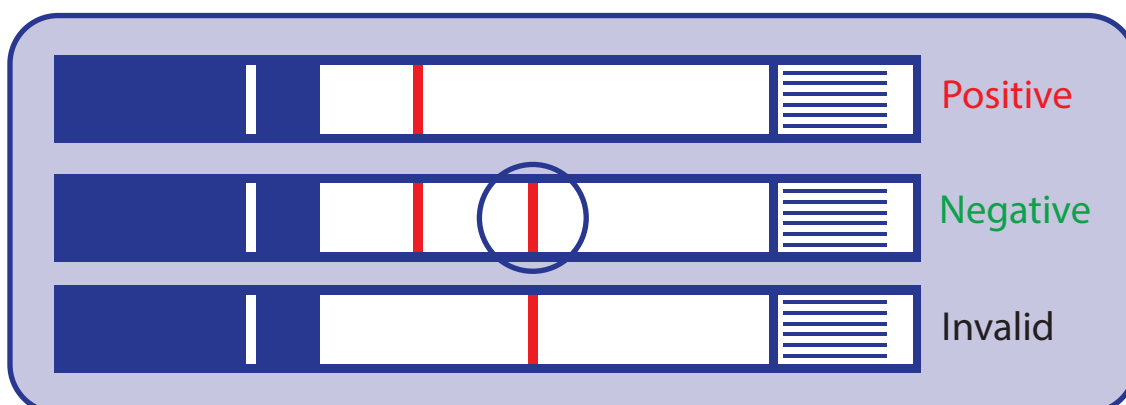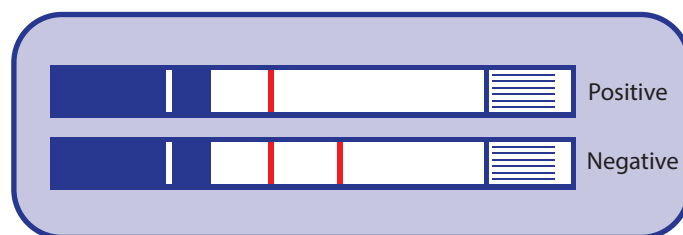

NOTE: It should be considered a **NEGATIVE** test even if a second red line is faded or faint
If possible, use an additional test to confirm your results



How to test for Fentanyl

IV Users Guide

Lower Your Risk of Overdose

Be Careful

- Go slow by taking smaller amounts
- Use with people you trust and inject at different times
- Have Naloxone available and know how to use it



Step 1: Testing Cooker Residue

Prepare Shot


- Prepare your shot as normal and set aside the needle to test your substance left over in the cooker before injection
- Add enough water to fill your cooker. Stir if necessary



Step 2: Testing

Use the Test Strip



- Place the test strip wavy side down in the water for 15s. Make sure the water is up to the line
- Remove strip from water and place it on a flat surface for 1-2 minutes

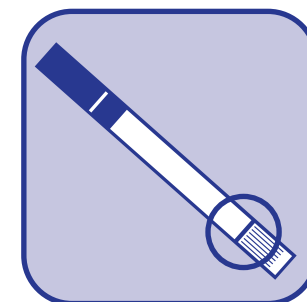

Step 3: Results

Read the Results

- A test **POSITIVE** for fentanyl is indicated by **ONE RED LINE**
- A test **NEGATIVE** for fentanyl is indicated by **TWO RED LINES**
- Remember, no test is 100% accurate, & a negative test does not necessarily mean you are safe

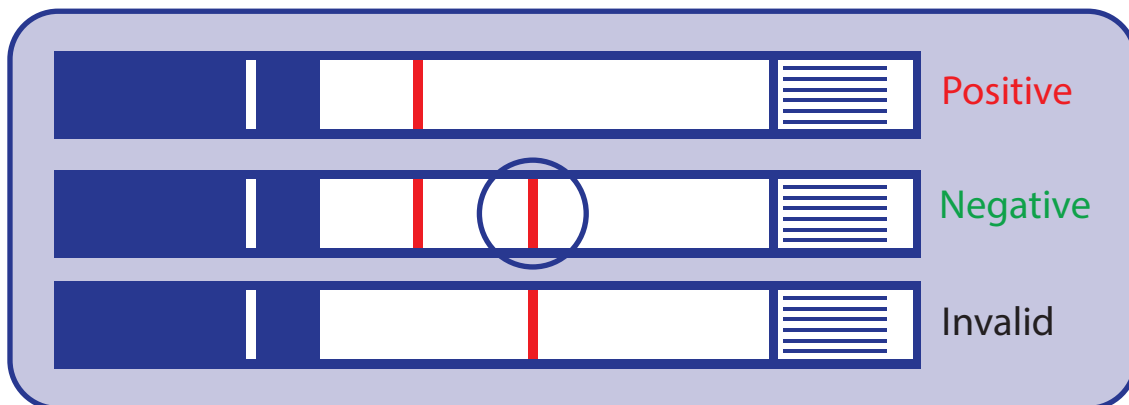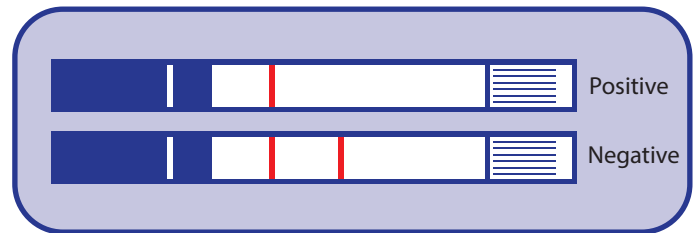

NOTE: It should be considered a **NEGATIVE** test even if a second red line is faded or faint
If possible, use an additional test to confirm your results

